Gebrauchsanleitung.....	3
Instructions for Use.....	21
Mode d'emploi.....	37
Manuale di utilizzazione.....	55
Gebruiksaanwijzing.....	73
Návod k použití.....	91
Návod na použitie.....	108
Instrucțiuni de utilizare.....	125
Instrukcje użytkowania.....	143
Instrucciones de uso.....	161

TENS-Pads

Arthro-Pads

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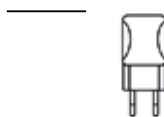
3. Cable for power adapter

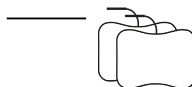
4. Remote control

5. Connection cable for TENS pads (Identifiable by the white cable end piece)

6. Arthro-Flow electrode pads for knee arthritis pain


7. Connection cable for Arthro pads (Identifiable by the blue cable end piece)


OVERVIEW AND DIAGRAMS

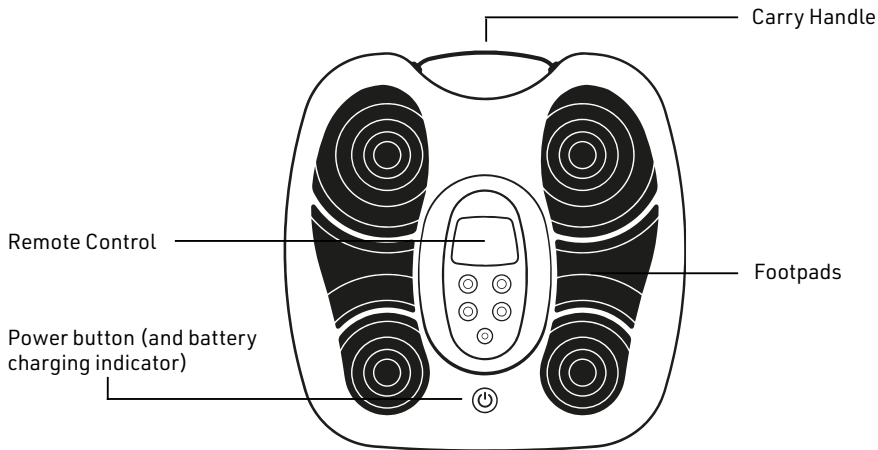Fflow base unit with remote in cradle

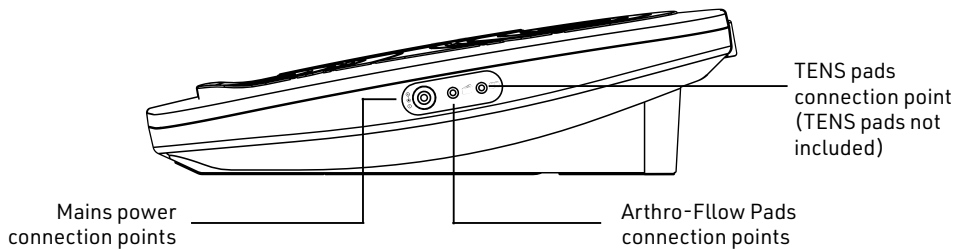Fflow base unit - side view of connection points


Fflow base unit - Reverse

Pull-out legs/
Relax and Recline mode


NOTE: Buttons allowing the legs to be set back into the device are located in the recessed channel that becomes visible when the legs are pulled out

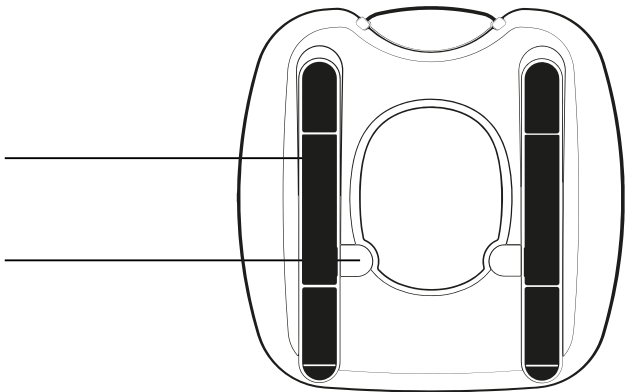
Access-point for pull-out legs

Remote Control

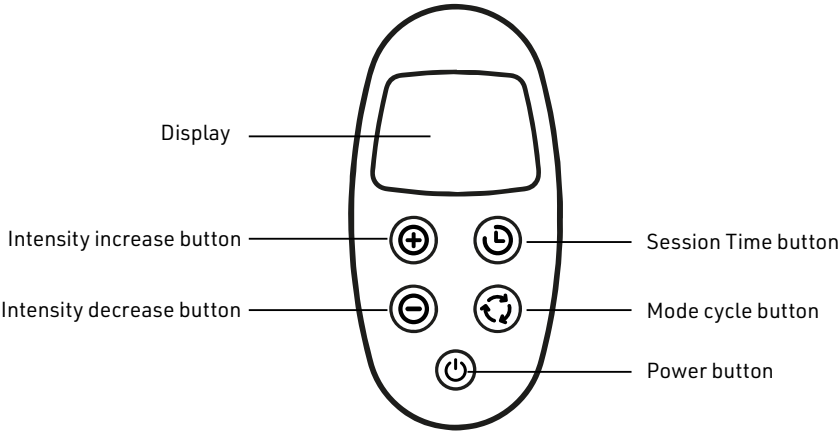Remote Control - Display Detail

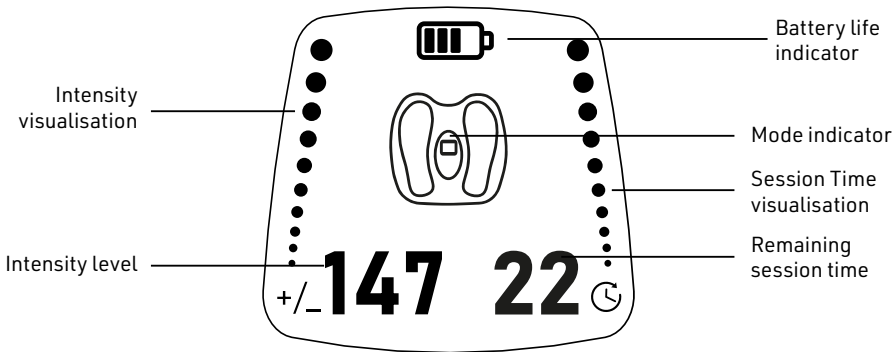INDICATIONS FOR USE

What is Fflow?

Fflow is a class IIa medical device that delivers electrical stimulation to the feet and legs.

What is Fflow used for?

Fflow is used to temporarily:

- Increase the blood circulation of the feet and lower legs.
- Relieve pain associated with sore or aching muscles in the feet and legs due to strain from exercise and regular household and work activities.
- Reduce discomfort linked to poor circulation, reduce swelling and feelings of tiredness and heaviness of the feet and legs.

Target Users

Adults only. Those with blood circulation disorders that affect the feet and/or lower legs and those suffering from associated pain and discomfort.

What are Fflow's different modes for?

Footpads

For electrical stimulation via the soles of the feet, which can trigger automatic contraction of the calf muscles.

Increases blood flow in the feet and lower legs and temporarily relieves pain and discomfort in the feet and legs linked to poor circulation.

Arthro-Flow Electrode Pads

Deliver electrical stimulation to the muscles in the upper legs. The Arthro-Flow pads are used to strengthen muscles that help support the knee and to provide temporary relief of knee joint pain associated with osteoarthritis of the knee.

TENS electrode pads (Not included in delivery – sold separately)

Can be placed on various parts of the body where they deliver electrical stimulation to the nerves beneath the skin. Provides temporary relief of pain associated with sore and aching muscles and joints in the neck, back, shoulder, arm, leg, and hip.

ACHTUNG! Flow is not intended to replace exercise or your current medication programme and its benefits are temporary.

READ ALL INSTRUCTIONS CAREFULLY BEFORE USING FLOW.

CONTRAINDICATIONS

Do NOT use Flow in the following circumstances:

- If you have suspected or confirmed epilepsy or heart disease, malignant tumour, serious cerebrovascular disease or other acute disease, unless specialist medical authorisation has first been obtained.
- If suffering from haemorrhage after acute trauma or fractures, or if recovering from surgery.
- If you are pregnant.
- If you are fitted with a pacemaker or other active implanted electrical device.
- On children or people with no ability to express their own consciousness.
- If you have been diagnosed with, or have any reason to believe that you may be suffering from, Deep Vein Thrombosis ("DVT").

DVT may be asymptomatic or may manifest the following symptoms:

- Throbbing or cramping pain in one leg (rarely both legs), usually in the calf or thigh swelling in one leg (rarely both legs).
- Warm skin around the painful area.
- Red or darkened skin around the painful area.
- Swollen veins that are hard or sore when touched.

Know the signs and consult with a doctor if you have any hesitations.

- Do NOT use if you have, in the area being treated, active or suspected cancer or undiagnosed pain with a history of cancer. Stimulation directly through a confirmed or suspected malignancy should be avoided as it may stimulate growth and promote spread of cancer cells.

WARNINGS

Consult with a doctor before using Flow in the following circumstances:

- You have very sensitive skin.
- You have abnormal blood pressure.
- You have heart rhythm problems.
- You have a pacemaker.
- You have had recent knee surgery.
- You are in the care of a doctor, or have received medical or physical treatment for your condition.
- You are unsure whether Flow is suitable for your condition.
- If electrical stimulation is applied over or close to a metal implant it may cause discomfort. If this occurs, reduce the level of intensity of the Flow device. If discomfort continues or worsens cease use and seek the advice of a doctor.
- Do not use Flow in close proximity (less than 1 meter) to other electrical medical equipment.
- If any of your symptoms worsen while using Flow, cease use immediately and seek the advice of a doctor.
- The user is the intended operator.
- Do not attempt maintenance or repair during use. Risk of harm.
- Do not refit or disassemble device. Risk of harm.
- Dust and lint may affect the performance of the unit, please use a dry soft cloth to clean the device as needed.
- Check that electrodes are firmly attached before each use to avoid adverse effects on performance and other issues.
- This product contains cables. Keep away from children to avoid risk of strangulation.
- If, contrary to expectations, you experience pain or discomfort, discontinue use immediately.

Warnings for the Arthro-Flow pads and TENS pads (TENS pads not included – sold separately)

- Do not apply the pads on the head, face, front of neck, or on the chest.
- Do not place the pads on broken skin. The pads

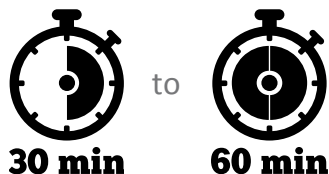
could encourage infection.

- Do not apply on sensitive or irritated skin, or on areas of skin that lack sensation.
- Pads are intended for one individual user only. Do not share pads between users
- Only use authentic Fflow-branded pads with the Fflow device, and do not use pads outside of their indicated use-by date
- Do not use Arthro-Flow pads or TENS pads with any other devices.
- If you experience any swelling (including joint swelling or tendonitis) as a result of using Fflow, discontinue use.
- Those with very sensitive skin may experience irritation or burns at the point of pad application
- Ensure that the device and remote control are within easy reach during operation, so that the device may be shut down promptly if needed (for example, if discomfort is experienced during use)
- If the user is using high frequency surgical equipment and this stimulator at the same time, there is a risk of burns at the point where the electrode pad is applied or damage to the device itself
- Operation in close proximity (e.g. 1 m) to shortwave or microwave therapy Medical Equipment may produce instability in the output of the device
- Application of electrodes near the thorax may increase the risk of cardiac fibrillation
- Do not stand on the Fflow device. Use only while sitting down or while reclining
- Do not immerse your device in water, spill liquid on it, or place it close to excessive heat or sources of high humidity
- Do not attempt to open or modify the device
- Keep out of the reach of children

OPTIMISING YOUR SESSIONS WITH FLOW


Footpad stimulation

For best results use the footpads for between 30 and 60 minutes every day. This is enough time to achieve therapeutic benefits. You can use the device more, if desired, up to 3 hours total in a day.

Arthro-Flow pads for arthritis

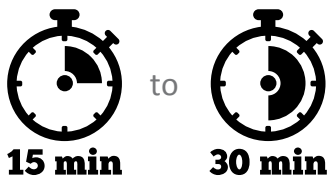Use the Arthro-Flow pads for arthritis for between 15 and 30 minutes, 5 days per week.

TENS pads (Not included in delivery - sold separately)

Use the wired TENS pads as and when required for up to 30 minutes per session.

USING YOUR FLOW DEVICE FOR THE FIRST TIME

Remove all parts from their packaging. Remove all packaging materials from the remote control. Insert the remote control into the charging station on the device.

IMPORTANT!: The very first time you use Fflow, you need to connect the device to the mains power supply. This will charge the battery, allowing you to use the device wirelessly in subsequent sessions.

We recommend charging the Fflow device and remote control for at least 2 hours before using it for the first time. Locate the power adaptor and power cable. Take the cable and locate the rectangular shaped USB connector at one end. Connect this into the power adapter. Now take the small, round connector at the other end of the cable. Connect this to your Fflow device. The connection point is found on the right-hand edge of the device and has this symbol next to it:

The connection is magnetic and will snap into position. Now plug the power adapter at the other end of the cable into a nearby electrical socket.

NOTE: Whenever your Fflow is connected to the mains power, the batteries in the device will automatically begin to charge. If the remote control is placed in the cradle the remote will also charge.

Turning the power on

Press the power button on the Fflow device. The power button on the device will light up blue and you will hear a short musical motif.

Pick the remote control up from its cradle in the device and press the power button on the remote. The display will activate and light up.

USING MODE 1 – STIMULATION VIA FLOW'S FOOTPADS

What is this mode for?

Use this mode to deliver stimulating pulses through the footpads to your feet and lower legs.

Take the remote control in hand. In the centre of the screen you will see this symbol.

This indicates that you are in mode 1: stimulation via the footpads. While seated, place both your bare feet onto the footpads of the device. Tap the „+“ button once to increase the intensity by one level. To increase the intensity more rapidly, hold the button down.

You should soon begin to feel the stimulation in your feet and lower legs. If the intensity becomes too strong, press the „-“ button to decrease intensity.

Aim to reach a level of intensity that is strong but

still comfortable for you. At this level the muscles in your lower leg will contract spontaneously in response to the stimulation. As you become familiar with the Fflow therapy you should find you are able to tolerate higher levels of intensity.

NOTE: that the feeling of stimulation will vary as Fflow progresses through its programme of stimulating pulse patterns.

When you have found your desired level of intensity, simply relax and let Fflow run through its program of stimulating pulse patterns. If you feel discomfort at any point during the session or wish to immediately terminate the Fflow therapy, remove one or both feet from the device.

Tip: You can apply moisturiser or a conductive gel to the soles of your feet to maximise the intensity of the therapy.

NOTE: Fflow does not move or vibrate. Both feet must be placed on the footpads for stimulation to be felt. If both feet are not making firm connection with the foot pads, Fflow will automatically reduce the intensity to a low level. This is a safety feature. To resume, place both feet firmly on the footpads. Fflow will automatically increase the intensity back to its former level.

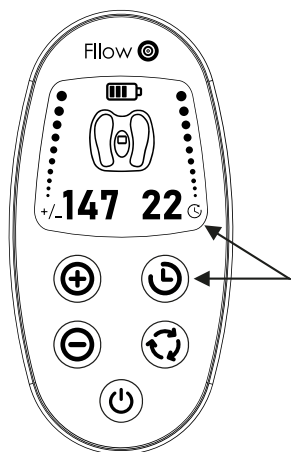
Viewing and Adjusting the session time

Each session with Fflow is automatically set to last for 30 minutes. The remaining minutes of the session are displayed on the right-hand side of the remote control screen.

You can select a different length of session by pressing the timer button on the remote control.

This will cycle through different options. You can select 15, 30, 45 or 60 minutes.

Mode 1

Finishing a session

Fflow will automatically turn itself off when the session timer reaches 00.

To finish the session at any time before this, press the power button on either the remote control or on the Fflow device. The stimulation through the footpads will cease and the device will shut down.

USING MODE 2 - STIMULATION VIA THE ARTHRO-FLOW PADS FOR THE KNEE

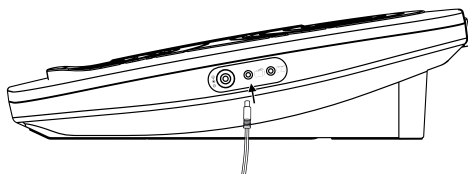
What is this mode for? This mode delivers strong electrical pulses via the large wired Arthro-Flow pads. The pads are placed on the legs, where they help to contract the muscles in the leg. The stimulation can help reduce sensations of pain around the knee joint.


ATTENTION!: Do not apply the Arthro-Flow pads to the head, neck or chest.

Connecting the Arthro-Flow pads

Remove the large, curved Arthro-Flow pads from their plastic bag packaging. Locate the connecting cable for the Arthro-Flow pads. This can be identified by a blue coloured connecting part at one end of the cable. Hold the blue end of the cable and connect this into the Fflow device. The connection point has this symbol next to it.

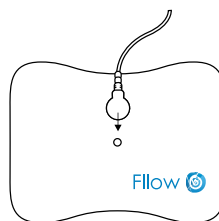At the other end of the cable are two connectors. Take one of these and connect it to one of the Arthro-Flow pads. The cable connects to the metal stud on the surface of the Arthro-Flow pad. Press the connection together firmly and it will snap into place.

Repeat this process with the other Arthro-Flow pad.

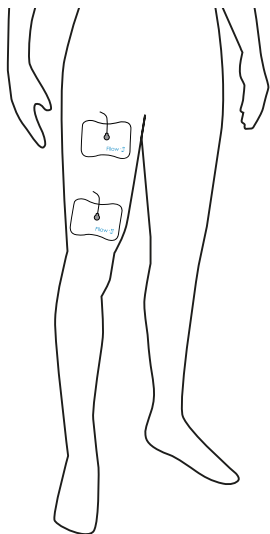
Applying the Arthro-Flow pads

Before use, clean the site of application (upper thigh and above the knee) with a damp cloth. Then dry thoroughly.

IMPORTANT!: Both Arthro-Flow pads must be applied to the same leg. The program is run on one leg at a time.

Remove the protective plastic liner from the back of one pad. Apply this on your upper thigh. Remove the protective plastic liner from the back of the second pad. Apply this pad above the knee of the same leg. See image for illustration. Retain the plastic liners for later.

Activating Mode 2: Arthro-Fflow pads

Press the power button on the Fflow device. The device will light up and you will hear a short musical motif.

Remove the remote control from its cradle and press the power button on the remote.

Now, press the cycle mode button ONCE.

In the centre of the screen you will see this symbol

This indicates that you are in mode 2: stimulation via the Arthro-Fflow pads for arthritis of the knee.

IMPORTANT! Ensure you are sitting down or reclining on a bed or couch before initiating next step.

Adjusting the intensity

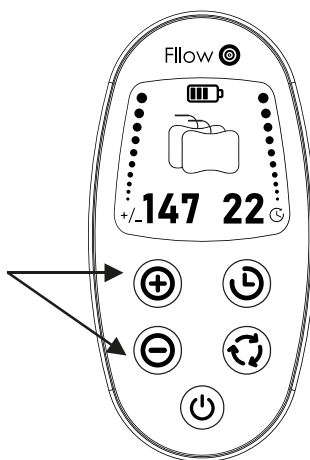
NOTE: The first few times you use this mode, be sure to set the intensity at a gentle, low level. Your muscles will likely need time to become accustomed to the stimulation.

Press the „+“ button on the remote to begin increasing the intensity of the stimulation. With each press of the „+“ button you will see the level

of intensity displayed on the screen increase. The intensity level is the number on the left side of the screen.

Increase the intensity only gradually until you begin to feel the stimulation in your thigh. This may feel like a tingling or similar sensation.

If the intensity becomes too strong, press the „-“ button to decrease the intensity level. Hold the button down to rapidly decrease the intensity. When you have found your desired level of intensity, relax and allow the programme to run its course.

Mode2

Over the course of several sessions, as you become familiar with the Arthro-Fflow pads, you should find you are able to tolerate higher levels of intensity. At higher levels you should begin to see a contraction of the thigh muscle. This is desirable and indicates that the stimulation is working correctly.

Viewing and Adjusting the session time

Each session with Fflow is automatically set to last for 30 minutes. The remaining minutes of the session are displayed on the right-hand side of the remote control screen.

You can select a different length of session by pressing the timer button on the remote control.

This will cycle through different options. You can select 15, 30, 45 or 60 minutes.

It is recommended to use the Arthro-Fflow pads for between 15 and 30 minutes per session.

Finishing a session

Fflow will automatically turn itself off when the session timer reaches 00.

To finish the session at any time before this, press the power button on either the remote control or on the Fflow device. The stimulation through the pads will cease and the device will shut down.

Removing the Arthro-Flow pads

Carefully remove each of the pads in turn from your body. Hold one corner of the pad and gently peel it back.

Tip: To extend the service life, we recommend cleaning the pads under warm, running water after each use. Replace the protective plastic liner onto the stick gel side of the pads. Then place both pads back into the zip lock bag they came in. This will help keep the pads in optimum condition for your next use.

Remove the connecting cable from the Fflow device.

USING MODE 3 – STIMULATION VIA THE TENS PADS (NOT INCLUDED IN DELIVERY - AVAILABLE SEPARATELY)

TENS Pads are available from: the free order hotlines **0800 400 66 50** (DE/AT) or **0800 400 665** (CH) and online at **www.mediashop.tv**

What is this mode for?

This mode delivers electrical pulses via the small, square, wired TENS pads. The pads can be placed on various parts of the body where they deliver TENS therapy, a form of electrical stimulation understood to have pain relieving or pain blocking effects.

Potential areas to apply the TENS pads to include the legs, hips, lower back, upper back, arms, and shoulders.

ATTENTION! Do not apply the TENS pads to the head, front or side of the neck, or the chest.

Back of shoulder

Upper back

Outside of foot

Knee

Lower back

Thigh

Inside of foot

Calf

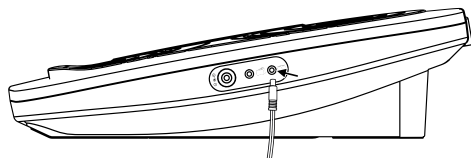
Connecting the TENS pads


Remove the two small, square TENS pads from their plastic bag packaging.

Locate the connecting cable for the TENS pads. This can be identified by a white-coloured connecting part at one end of the cable.

Hold the end of the cable where the white connecting part is located and connect this end into the Flow device. The connection point has this symbol next to it.

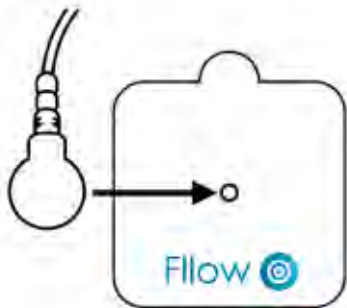At the other end of the cable are two connectors. Take one of these and connect it to one of the TENS pads. The cable connects to the metal stud on the surface of the TENS pad. Press the connection together firmly and it will snap into place. Repeat this process for the second TENS pad.

Applying the TENS pads

Before use, clean the site of application with a damp cloth. Then dry thoroughly. Take one of the TENS pads in hand and remove the protective plastic liner from the back of the pad. Apply this on the desired area. Now remove the protective plastic liner from the back of the other TENS pad. Apply this on the desired area, at least one inch away from the other pad. Retain the plastic liners for later.

NOTE: To ensure correct use and functioning of the device, both TENS pads must always be attached to the body.

Activating Mode 3: TENS pads

Press the power button on the Flow device. The device will light up and you will hear a short musical motif. Remove the remote control from its cradle and press the power button.

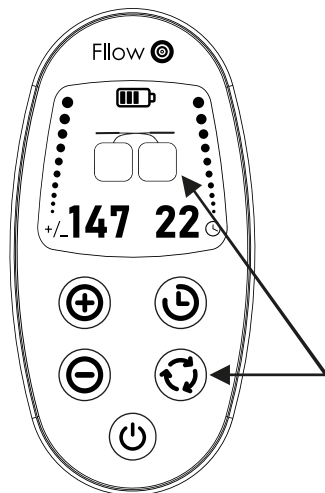
Now, press the cycle mode button TWICE.

In the centre of the screen you will see this symbol.

This indicates that you are in mode 3: stimulation via the TENS pads for pain relief.


Mode 3

Adjusting the intensity

Press the „+“ button on the remote to begin increasing the intensity of the stimulation. With each press of the „+“ button you will see the level of intensity displayed on the screen increase. The intensity level is the number on the left side of the screen.

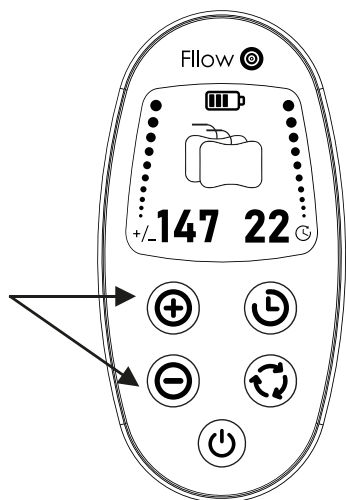Tap the „+“ button once to increase the intensity by one level. To increase the intensity more rapidly, hold the button down. Increase the intensity gradually. You should soon begin to feel the stimulation in the area you have placed the TENS pads. Aim to reach a level of intensity that is strong but still comfortable for you. Over the course of several sessions, as you become familiar with the Flow therapy, you should find you are able to tolerate higher levels of intensity that help to maximise the effects of the stimulation. If the intensity becomes too strong, press the „-“ button to decrease the intensity level. Hold the button down to rapidly decrease the intensity. When you have found your desired level of intensity, simply relax and let Flow run through its programme of stimulating patterns.


Viewing and Adjusting the session time

Each session with Flow is automatically set to last for 30 minutes. The remaining minutes of the session are displayed on the right-hand side of the remote control screen.

You can select a different length of session by pressing the timer button on the remote control.

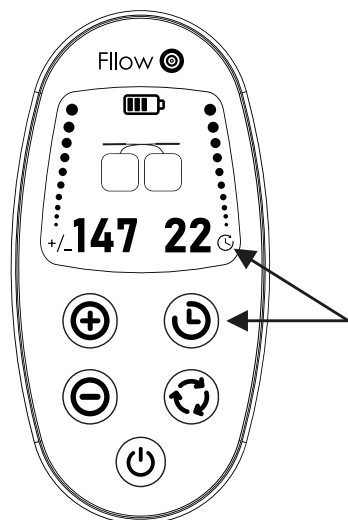This will cycle through different options. You can select 15, 30, 45 or 60 minutes.

Finishing a session

Flow will automatically turn itself off when the session timer reaches 00.

To finish the session at any time before this, press the power button on either the remote control or on the Flow device. The stimulation through the pads will cease and the device will shut down.

Removing the TENS pads

Carefully remove both of the TENS pads in turn from your body. Hold one corner of the pad and gently peel it back.

Tip: To extend the service life, we recommend cleaning the pads under warm, running water after each use. Replace the protective plastic liner onto the sticky gel side of the pads. Then place both pads back into the zip lock bag they came in. This will help keep the pads in optimum condition for your next use.

Remove the connecting cable from the Flow device.

MULTI-MODE APPLICATION

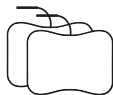
Flow allows you to apply multiple modes simultaneously, depending on your preference:

1. Footpad stimulation

2. Arthro-Flow pads stimulation

3. Tens pads stimulation

1. Press the MODE CYCLE button until you reach the mode you want to apply first. Follow the instructions in the manual above for that mode until you have set your desired level of intensity.
2. Now press the MODE CYCLE button on the remote control.
3. The symbol in the centre of the remote control screen will change to show the newly selected mode. You can now begin setting the intensity for this mode.
4. Press the MODE CYCLE button again to select the third possible mode. You can now set the intensity for this mode.
5. Pressing the MODE CYCLE button once more will bring you back to the first of the three modes.

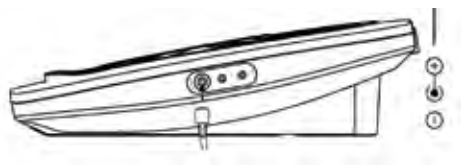
You can continue to cycle through modes and adjust the intensity settings as required.

NOTE: You cannot set different session times for multiple modes. All modes will finish when the on-screen timer displays 00.

CHARGING FLOW'S RECHARGEABLE BATTERY

Fflow can be run either from the mains power supply or via its internal rechargeable battery. When the battery inside the Fflow device is running low, the power symbol will turn orange. To charge Fflow's battery, connect the device to the mains power supply. Take the AC/DC power adaptor and locate the small, round connector at the end of the cable. Connect this to your Fflow device. The connection point is found on the right hand edge of the device and has this symbol next to it

The connection is magnetic and will snap into position. Now plug the power adaptor at the other end of the cable into a nearby electrical socket. The power button will flash to indicate that charging is in progress. The battery in Fflow will take approximately 2.5 to 3 hours to charge. When charging is complete the power button will turn solid blue. Fflow will deliver up to 36 hours of use on a full battery charge (dependent upon mode and intensity level selected).

Powering Fflow from its rechargeable battery

When Fflow's battery is charged you can run the device without connecting it to the mains power. Simply disconnect the power cable by pulling it gently away from the device. Fflow will automatically switch to battery power once the mains cable is disconnected (as long as there is sufficient charge in the battery).

CHARGING THE REMOTE CONTROL

Fflow's remote control is powered exclusively by its internal rechargeable battery.

IMPORTANT!: the battery in the remote control cannot be removed or replaced.

When the battery inside the remote control is running low, the battery indicator at the top of the screen will flash and show as empty.

To charge the remote, place it in the cradle in the centre of the Fflow device and connect the device to the mains power supply.

The remote control will automatically begin charging when Fflow is connected to the mains power. The bars of the battery symbol will progress to indicate that charging is in progress. The battery in the remote will take approximately 2 hours to charge. Once charging is complete the battery indicator will show as full with 3 black BARS inside it. The remote will provide up to 16 hours of use on a full battery charge.

Remote control in charging position in cradle

ACTIVATING FLOW'S RELAX AND RECLINE MODE

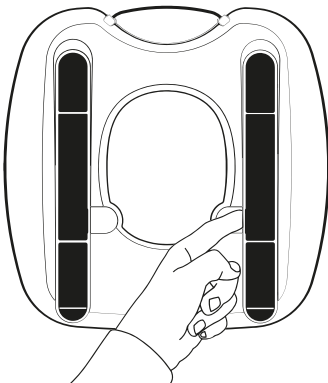
By activating FLOW's Relax and Recline Mode, you can use FLOW's footpad stimulation while reclining, for example on a sofa or a bed.

Place the device on your chosen surface (bed, sofa, etc).

Turn the device over and place it face down.

On the reverse of the device you will see two runners, in black rubber, as well as two small indents by the sides of the runners.

Place your finger in one of the indents and lift the runner up and away from the device until it locks in place. Repeat this process with the other runner. You will now have two supporting legs projecting away from the device.

Turn the entire FLOW device over again so that the foot pads are facing upward.

NOTE: To protect furnishings from potential marking place a towel or piece of fabric under the legs of the device.

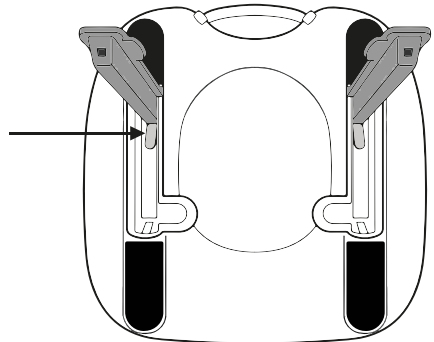
You can now use FLOW as normal, following the instructions further above in this manual. The increased angle of the device allows you to easily place your feet on it while fully reclining for greater relaxation and comfort.

To deactivate the 'Relax and Recline' mode, turn the entire device back over and return the support feet to their original position.

NOTE: Next to the joint of each supporting leg there is a small button. Press this button and simultaneously push the supporting leg back down toward the device until it lies flush against the device.

Replacing the Arthro-Flow pads and TENS pads

After a certain number of uses the pads will lose their adhesiveness. The number of applications for each gel pad will vary depending on the user's skin condition and humidity.

Replacement Arthro Pads and TENS Pads are available from: the free order hotlines **0800 400 66 50** (DE/AT) or **0800 400 665** (CH) and online at **www.mediashop.tv**

CLEANING AND MAINTENANCE

To clean, wipe Fflow with a lightly damp cloth. Ensure the device is switched off during cleaning. The arthro pads and TENS pads (not included) can be rinsed with warm water and then air-dried to extend their service life. Ensure that the

Arthro-Flow pads and TENS pads are always placed back onto their protective plastic covers when not in use. This will help to maximise the lifespan of the pads. If the pads no longer adhere to the skin on their own, they must be replaced.

TROUBLESHOOTING/PROBLEM SOLVING

Problem	Possible reason	Suggested Action
"My Fflow device is active (the LED in the power button is blue), but I cannot feel the stimulation through my feet"	The intensity level may be set too low.	Press the "+" button on the remote control to gradually increase the level of intensity. Or, hold this button down to rapidly increase intensity until you feel the stimulation coming through. Consult the number in the lower left corner of the screen to see the intensity level you currently have selected. Fflow offers 150 levels of intensity.
	Your feet may not be making firm connection with the footpads.	Ensure that footwear and items of clothing are removed from your feet. Firmly press both of your bare feet onto the footpads. Both your left and right foot must make contact with the pads in order for the device to deliver stimulation correctly.
	Your feet may be dry.	Stimulation via the footpads will be improved if the feet have some moisture. Ensure you are well hydrated when using the device. You can also apply conductive gel or moisturiser to the soles of the feet to increase the intensity of the stimulation.
	Another mode is selected.	The screen must be displaying the footpads symbol before you can adjust the intensity of the footpad stimulation. Press the Mode Cycle button once or twice until you see the footpad symbol displayed. 
		If none of the above steps is effective for you, please contact the point of purchase or the distributor for your territory.

The device will not turn on (the LED in the power button does not light up).	The device has no charge in its battery.	Connect Flow to the mains power using the AC/DC power adaptor. The LED will flash blue to indicate that the device is charging. The device can be used while it is charging.
	The power cable is not connected.	Check that power is switched on at the electrical socket. Then check that the power cable is connected to the device.
The intensity is rapidly dropping. I see the numbers for the intensity level falling on the remote control screen.	Both feet are not firmly placed on the footpads and creating a secure connection. In this case Flow automatically reduces the intensity level as a safety feature.	Place both feet more firmly on the footpads. Avoid moving feet while the session is in progress.
The remote control will not turn on.	Remote has no charge in its battery.	Place the remote control in its cradle in the centre of the Flow device. Connect the Flow device to the mains power, using the AC/DC adaptor. The battery indicator at the top of the screen will indicate that charging is in progress. You can use the device while the remote control is charging.
The device is at an uncomfortable angle.	The Relax and Recline system needs to be adjusted.	If using Flow while seated make sure the legs on the underside of the device are fully recessed. If using the device while reclining or lying down, make sure the legs are pulled out from the device.
The legs of the Relax and Recline system won't lower back into the device.	The lock-release buttons need to be pressed.	Under each leg there is a lockrelease button. Press this button and the leg should easily lower and fit back into the Flow device.
I cannot feel any stimulation through the Arthro-Flow pads for arthritis.	Pads are not connected.	Ensure the pads are properly connected to the device by the cable. The connecting cable should be inserted into the side of the device. The connection point has this symbol next to it. 


	Another mode is selected.	To adjust the intensity of the Arthro-Flow pads the screen must be displaying the Arthro-Flow pads symbol. Press the Mode Cycle button once or twice until you see this symbol. 
	Not both Arthro Pads are attached to the body.	To ensure that the device functions correctly, both pads must always be attached to the knee or thigh.
I cannot feel any stimulation through the TENS pads(not included – sold separately).	Pads are not connected.	Ensure the TENS pads(not included – sold separately) are properly connected to the device by the cable. The connecting cable should be inserted into the side of the device. The connection point has this symbol next to it. 
	Both TENS pads (not included – sold separately) are not attached to the body.	To ensure that the device functions correctly, both pads must always be attached to the body.
	Another mode is selected.	To adjust the intensity of the TENS pads(not included – sold separately), the screen must be displaying the TENS pads symbol. Press the Mode Cycle button once or twice until you see this symbol. 
My legs ache after treatment with the footpads.	The intensity setting may be too high.	Select a lower intensity setting when using the footpads, to prevent muscles in the legs from being overstimulated.
My legs ache after treatment with the Arthro-Flow pads for arthritis.	The intensity setting may be too high.	Select a lower intensity setting when using the Arthro-Flow pads, to prevent muscles in the legs from being overstimulated.
Stimulation with the Arthro-Flow pads is intermittent.	The therapy is in a rest period.	The Arthro-Flow pads deliver periods of stimulation followed by rest periods with no stimulation. The device is functioning as intended.

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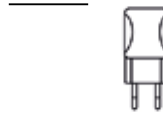Electronic Equipment Act (ElektroG) to contribute to the recycling of electrical and electronic equipment. Lamps and batteries, including rechargeable batteries, that are in a discharged state, are not permanently installed in the device and can be removed without causing damage, must be disposed of separately. Use rechargeable batteries instead of single-use disposable batteries wherever possible. The reuse and recycling of old devices contributes significantly to protecting our environment. If disposed of improperly, toxic substances contained in the device may be released into the environment and have adverse health effects on people, animals, and plants. Batteries with an increased pollutant content are also marked with the following symbols: Cd = cadmium, Hg = mercury, Pb = lead. Consumers are responsible for deleting personal data from the device themselves. The following are some tried and tested methods and recommendations for extending the service life of your batteries and rechargeable batteries and enabling their reuse:

- Charge your rechargeable batteries correctly and fully to maximise their service life. If necessary, discharge them completely using a suitable charger before recharging them.
- Consider the possibility of using rechargeable batteries. They can be reused and recharged many times, which reduces the amount of battery waste.
- Always use the correct type of battery for your devices. Incorrect use can shorten the service life of the batteries and possibly lead to harmful effects.

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Shenzhen Kentro Medical Electronics Co. Ltd.
Share Info GmbH
CH: MediaShop Schweiz AG | Leuholz 14 | 8855 Wangen | Switzerland
EU: MediaShop GmbH | Schneiderstraße 1, Top 1 | 2620 Neunkirchen | Austria